




Instructions for Preserving Eyewear/Face Shields



- 
- Remove gown and gloves and perform hand hygiene.
 - After exiting patient room, put on clean gloves.
 - Remove eye protection/face shield.
 - Remove any visible soil.
 - Use disinfectant to wipe down the eyewear.
 - Perform hand hygiene.
 - Use an alcohol pad to remove any residue left from the disinfectant after it dries.
 - *Do not touch eyewear while providing patient care. If eyewear is touched immediately perform hand hygiene.*
 - Discard when the field of vision is obscured or the product is damaged.

Note: Straps do not need to be disinfected routinely unless you know that it had been contaminated with patient secretions.